INNOVATIVE DESIGNS
for independent living™

DN7075 DUAL TUB GRIP INSTALLATION INSTRUCTIONS AGARRE DOBLE PARA TINA INSTRUCCIONES DE INSTALACIÓN POIGNÉE DOUBLE DE BAIGNOIRE DIRECTIVES D'INSTALLATION

Warning ! – FOLLOW THESE INSTALLATION INSTRUCTIONS CAREFULLY. PROPER INSTALLATION IS THE INSTALLER'S RESPONSIBILITY. FAILURE TO FOLLOW CORRECT INSTALLATION PROCEDURES CAN CAUSE THE TUB GRIP TO BECOME LOOSE, WHICH CAN RESULT IN SERIOUS INJURY .

Specifications:

Fits on most standard tub configurations. Install only on tubs with flat parallel walls. Do not install on tubs with sliding shower door tracks. For use on or with tub walls from 3" thick to 5.25" thick. Wipe tub wall clean and dry before installation. Check bar periodically after installation to make sure clamps are secure.

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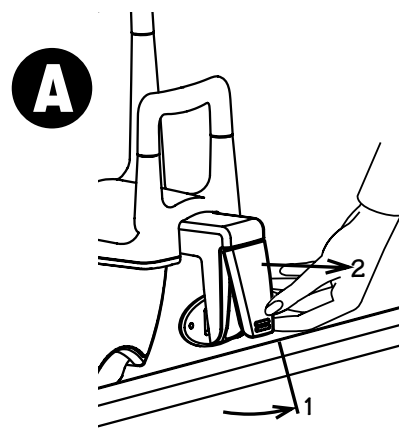
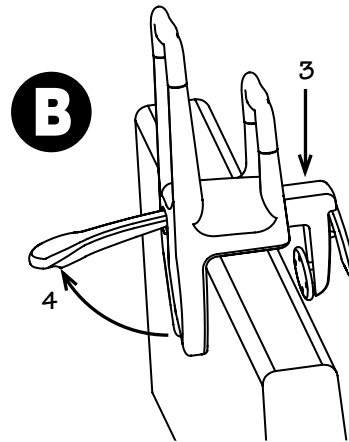
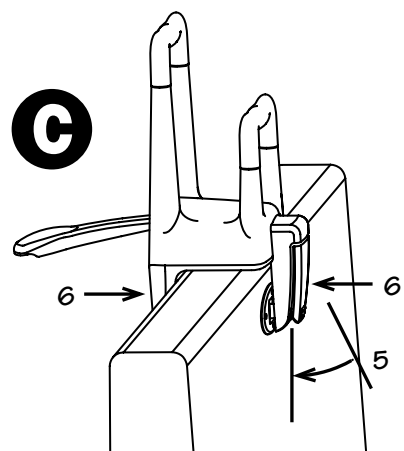
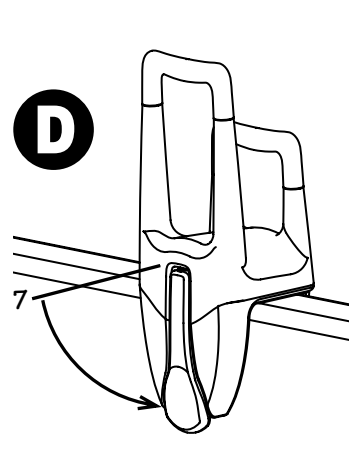
INNOVATIVE DESIGNS
for independent living™

WARNING:

TO PREVENT SERIOUS OR FATAL INJURY: FOR USE ONLY BY PERSONS WEIGHING LESS THAN 350 POUNDS. Consult healthcare professional for advice and instructions specific to your needs before use. Check bar periodically to be sure all clamps are secure. See instructions for additional information. Contact CSI immediately at 1-800-882-0116 if any components are missing or damaged. Assemble in accordance with CSI's instructions. Do not make substitutions or modify this product in any way.
FOR QUESTIONS OR ASSISTANCE WITH INSTALLATION CONTACT CSI AT 1-800-882-0116.

LIMITED LIFETIME WARRANTY

If this product fails due to a defect in materials or workmanship at any time during the life of the product, Creative Specialties International, a division of Moen Incorporated (CSI) will replace it free of charge, postage-paid. Simply contact CSI at 1-800-882-0116 for replacement information. This warranty does not cover products which have been abused, altered, damaged, misused, cut or worn. TO THE EXTENT PERMITTED BY LAW, CSI DISCLAIMS ALL OTHER IMPLIED OR EXPRESS WARRANTIES INCLUDING ALL WARRANTIES OF MERCHANTABILITY AND/OR FITNESS FOR A PARTICULAR PURPOSE.

LIMITATION OF LIABILITY: THIS WARRANTY IS YOUR SOLE REMEDY AND CSI SHALL NOT BE LIABLE FOR ANY DAMAGES, WHETHER DIRECT, INDIRECT, INCIDENTAL, SPECIAL, CONSEQUENTIAL, EXEMPLARY, OR OTHERWISE, INCLUDING LOST REVENUES AND LOST PROFITS, ARISING OUT OF ANY THEORY OR RECOVERY, INCLUDING STATUTORY, CONTRACT OR TORT. NOTWITHSTANDING THE TERM OF ANY LIMITED OR IMPLIED WARRANTY, OR IN THE EVENT THAT ANY LIMITED WARRANTY FAILS OF ITS ESSENTIAL PURPOSE, IN NO EVENT WILL CSI'S ENTIRE LIABILITY EXCEED THE PURCHASE PRICE OF THIS PRODUCT. SOME STATES AND PROVINCES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGES SO THE ABOVE LIMITATIONS OR EXCLUSIONS MAY NOT APPLY TO YOU. THIS LIMITED WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS, AND YOU MAY ALSO HAVE OTHER RIGHTS WHICH VARY FROM STATE TO STATE AND PROVINCE TO PROVINCE.

For additional warranty information, visit www.homecare.moen.com.

The Home Care Dual Tub Grip meets or exceeds the applicable requirements of the following standards / guidelines:

The Americans with Disabilities Act (ADA)

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Creative specialties is a division of Moen Incorporated. © 2005
Home Care is a registered trademark of Home Care Tim-BR-MART, licensed to Moen Incorporated.
Home Care and Creative Specialties are registered trademarks of Moen Incorporated.